



PRO WHITENING

BOOST

WHITEN / RECOVER / BOOST

(S) PRO WHITENING

BOOST

Saber White Boost is a professional-grade rapid whitening treatment designed to offer quick results on the go.

Before treatment: Brush and floss between teeth.

1



Twist the base of the pen to release gel

2



Paint a thin layer onto the front (smile zone) of your teeth

3



Place the LED light between your teeth & press the power button. Leave in for 10 minutes

4



Remove light & rinse mouth with warm water

Do not use if:

- You are pregnant or breastfeeding
- You are allergic to any of the ingredients
- You have periodontal disease

Caution:

Saber White Boost contains 6% hydrogen peroxide. Prolonged contact may cause irritation. Rinse affected areas thoroughly with water. In extreme cases please consult your doctor. Please do not swallow the gel. Consult your dentist if you are taking any mouth or throat medication.

Ingredients - Whitening Gel: Hydrogen Peroxide, Glycerin, Propanediol, Carbopol, Menthol, Triethanolamine



Irritant



Keep Refrigerated



Recycle Packaging

Keep out of reach of children.

Manufactured by Dinamo Ltd, 1 The Overcliffe, Kent DA11 0EF

saberwhite.com