




**PRO** WHITENING

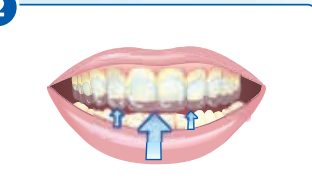
WHITEN / RECOVER / BOOST


# (S) PRO WHITENING

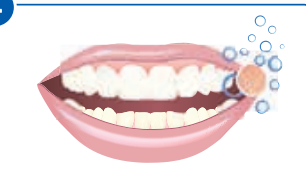
Saber White is a professional-grade treatment designed to whiten discoloured and stained teeth.

**Before treatment:** Brush and floss between teeth.

- 

**1** Apply gel to tray using syringe & plunger provided  
(Careful not to overfill the tray)
- 

**2** Insert tray onto teeth & wipe excess gel from gums
- 

**3** Leave for 1 hour  
or as directed by your dentist  
(Do not eat, drink or smoke during treatment)
- 

**4** Remove tray & chew 1 recovery tablet for 30 seconds  
(Just spit, no need to rinse)

**Ingredients - Whitening Gel:** Carbamide Peroxide, Glycerine, Propanediol, Carbopol, Menthol, Triethanolamine

**Ingredients - Recovery Tablets:** Sodium Bicarbonate, Tartaric Acid, Citric Acid, Chalk, Kaolin Clay, Xylitol, Magnesium Stearate, Polyethylene Glycol, Sodium Cocoyl Isethionate, Hydroxylapatite, Gelling Agent [Bark Extract], Flavouring, Sodium Monofluorophosphate, Sodium Benzoate

**Fluoride Content:** Contains Sodium Monofluorophosphate 1.1%w/w (1450ppm F)



Irritant



Keep Refrigerated



Recycle Packaging

Keep out of reach of children.

Manufactured by Dinamo Ltd, 1 The Overcliffe, Kent DA11 0EF

## Do not use if:

- You are pregnant or breastfeeding
- You are allergic to any of the ingredients
- You have periodontal disease

## Caution:

Saber White contains 16% carbamide peroxide. Prolonged contact may cause irritation. Rinse affected areas thoroughly with water. In extreme cases please consult your doctor. Please do not swallow the gel. Consult your dentist if you are taking any mouth or throat medication.

[saberwhite.com](http://saberwhite.com)